INOCOS

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High Temperature Steriliser

EN

THANKS FOR PURCHASING THE HIGH TEMPERATURE STERILISER INOCOS.
THIS DEVICE IS EXCLUSIVELY FOR PROFESSIONAL USE.

BEFORE OPERATING THIS EQUIPMENT, CAREFULLY READ ALL INFORMATION CONTAINED IN THIS HIGH TEMPERATURE STERILISER INFORMATION MANUAL AND KEEP IT FOR FUTURE REFERENCE.

THE INOCOS HIGH TEMPERATURE STERILIZER IS A **DRY HEAT STERILIZATION EQUIPMENT** INTENDED FOR BEAUTY AND HAIRDRESSING INSTRUMENTS AND CAPABLE OF **ELIMINATING MICROORGANISMS**. ITS USE IS RECOMMENDED FOR **STERILIZATION OF METAL INSTRUMENTS** AND IS NOT RECOMMENDED FOR INSTRUMENTS WITH PLASTIC, RUBBER OR WOOD COMPONENTS.

01 Components

AFTER OPENING THE BOX, CHECK IF THE FOLLOWING ITEMS ARE INCLUDED: **HIGH TEMPERATURE STERILISER WITH REMOVABLE TRAY, WITH POWER CABLE, TWO METAL RINGS, TWO EXTRA FUSES, INSTRUCTION MANUAL AND WARRANTY TERM.**

02 Caption

1. POWER CABLE ENTRY
2. LID
3. INDICATOR OF TIME (BLUE)
4. BUTTON REGULATOR OF TIME
5. INDICATOR OF TEMPERATURE (RED)
6. BUTTON REGULATOR OF TEMPERATURE
7. REMOVABLE ALUMINIUM TRAY
8. MANCHE EN BOIS

03 Use instructions

1. REMOVE ALL PROTECTIVE FILMS FROM THE INSIDE OF THE EQUIPMENT.
 2. CONNECT THE POWER CABLE TO THE APPLIANCE AND THEN CONNECT IT TO A POWER SOURCE. THE DIGITAL SCREEN WILL LIGHT UP.
 3. WASH THE INSTRUMENTS TO BE STERILIZED WITH SOAP AND WATER, AND DRY VERY WELL TO REMOVE THE RESIDUES.
 4. PLACE THE INSTRUMENTS IN THE STERILISER TRAY AND CLOSE THE LID.
- TIP:** TO PROTECT THE MATERIAL, WRAP THE TIP OF PLIERS, SCISSORS AND OTHER CUTTING INSTRUMENTS IN ALUMINUM FOIL BEFORE PUTTING TO STERILIZE.
5. PRESS THE RIGHT BUTTON AND TURN IT TO REGULATE THE TEMPERATURE. RECOMMENDED TEMPERATURE: 160°C.
 6. PRESS THE LEFT BUTTON AND TURN IT TO SET THE TIMER. RECOMMENDED TIME: 90 MINUTES. THE EQUIPMENT WILL BEAT WHEN THE CYCLE BEGINS AND ENDS. SLIGHT FLUCTUATIONS IN TEMPERATURE DURING THE STERILISATION PROCESS ARE NORMAL.

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